

NEW! Firmware Release: 3.5.0

Firmware release updates can be done via the GaitLab app allowing patients access to the latest features immediately!


✓ **NEW ACTIVITIES:**

1. Rehabilitation
2. Elliptical
3. Sprinting

✓ **ENHANCEMENTS:**

1. Cycling
2. Golfing

**ENGINEERED
TO EXCEED
EXPECTATIONS**

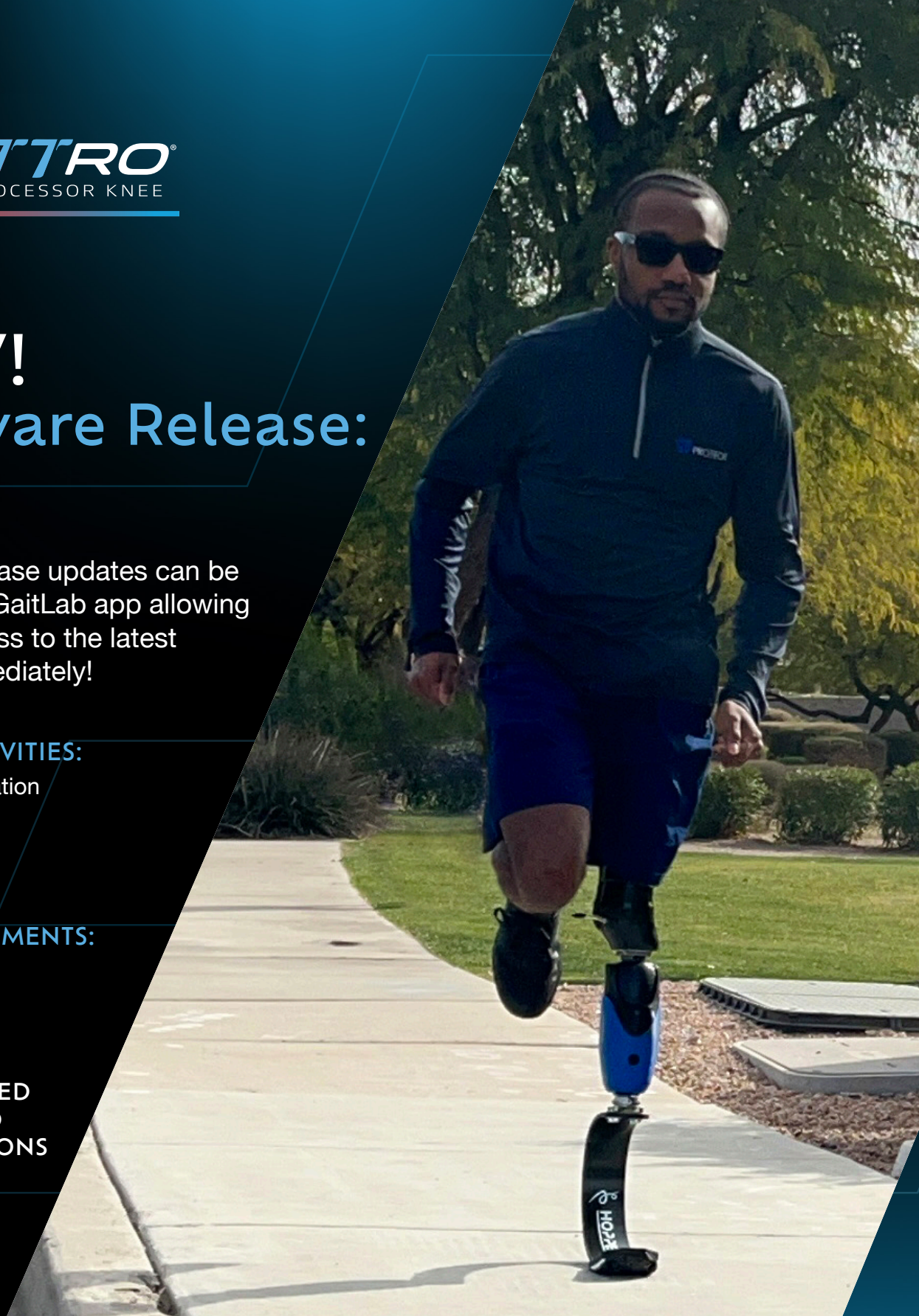

It's What's Inside That Counts

MORE Intuitive
For elevated security
and dependability

MORE Personalized
For a custom experience

MORE Adaptable
for fewer interruptions

NEW ACTIVITIES*

- ✓ **REHABILITATION**
Sit-to-Stand Support with adjustable resistance and timer
- ✓ **ELLIPTICAL**
Use of Elliptical machine with one adjustable flexion and extension resistance
- ✓ **SPRINTING**
Sprinting with multiple resistance adjustments

*Activities must be turned on in GaitLab app to be seen in Freedom Innovations app

LEARN MORE ABOUT 3.5.0

<https://www.proteorquattro.com/resources>



ENHANCEMENTS

- ✓ **CYCLING**
Obstacle Assist and Stance Lock are disabled for more predictable performance
- ✓ **GOLFING**
Stance Lock is disabled allowing patient to easily reset Golfing angle
- ✓ **UPDATE FIRMWARE**
 1. Connect to knee with GaitLab app
 2. Tap the "Tools" tab
 3. Tap "Update Firmware"
 4. Turn on Activities for patient use in Freedom Innovations app



PROTEOR
QUATTRO[®]
MICROPROCESSOR KNEE